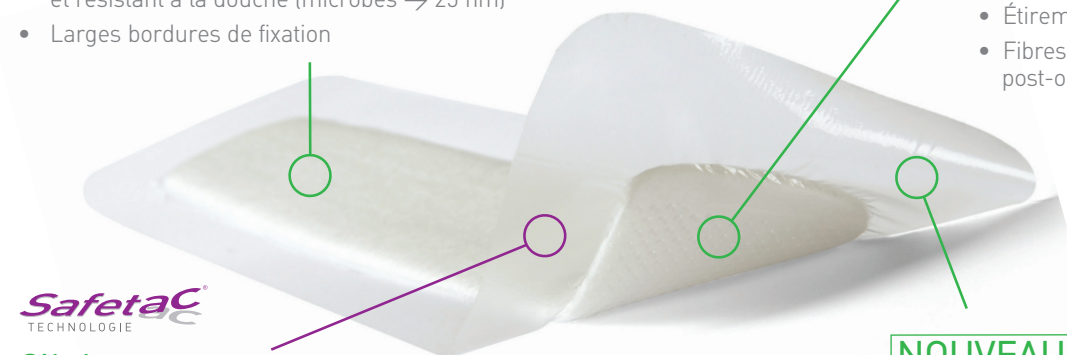
Mepilex® Border Post-Op

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Références :

1. Johansson C. et al. An assessment of a self-adherent, soft silicone dressing in post operative wound care following hip and knee arthroplasty. Poster presentation at EWMA, Brussels, Belgium 2012. 2. White R. A multinational survey of the assessment of pain when removing dressings. Wounds UK, 2008. 3. White R. et al. Evidence for atraumatic soft silicone wound dressing use. Wounds UK, 2005. 4. Meaurio S. et al. A study to compare a new self-adherent soft silicone dressing with a self-adherent polymer dressing in stage II pressure ulcers. Ostomy Wound Management, 2003. 5. Feili F. et al. Retention capacity. Poster presentation at the EWMA conference, Lisbon, Portugal 2008. 6. Wiberg A.B. et al. Preventing maceration with a soft silicone dressing: in-vitro evaluations. Poster presented at the 3rd Congress of the WUWHs, Toronto, Canada, 2008. 7. Davies P. et al. Evidence review: the clinical benefits of Safetac technology in wound care. Journal of Wound Care, 2008. 8. Santamaria N. et al. Clinical effectiveness of a silicone foam dressing for the prevention of heel pressure ulcers in critically ill patients: Border II Trial. Journal of Wound Care, 2015. 9. Santamaria N. et al. An estimate of the potential budget impact of using prophylactic dressings to prevent hospital-acquired PUs in Australia. Journal of Wound Care, 2014.

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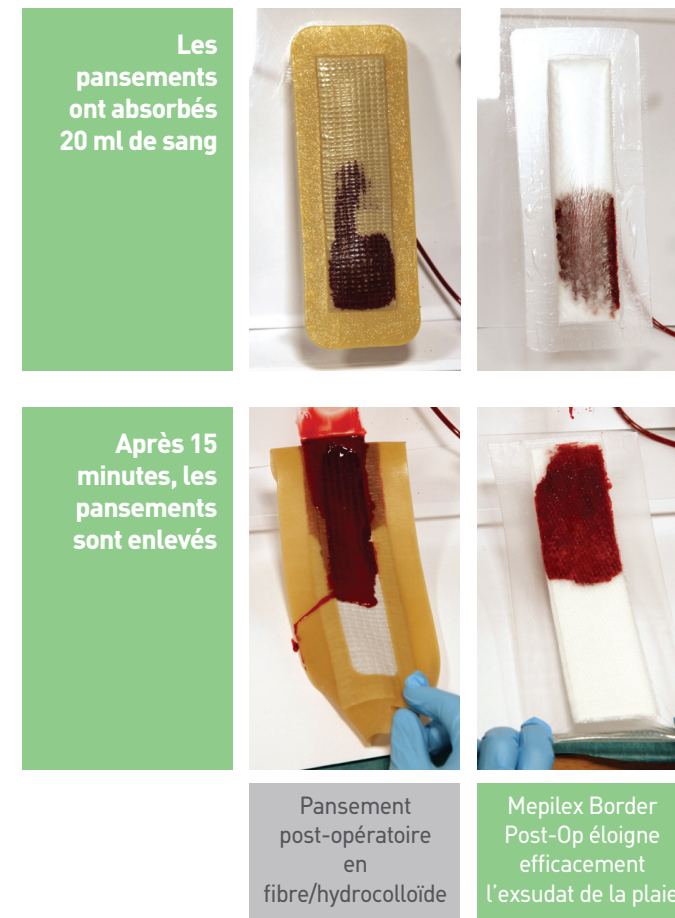
Références :

10. Johansson C. et al. Preventing post-operative blisters following hip and knee arthroplasty. Wounds International, 2012. 11. Van Overschelde, P. et al. A randomised controlled trial comparing two wound dressings used after elective hip and knee arthroplasty. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 12. Zarghooni K. et al. Effect of a modern dressing compared to standard dressings on outcome after primary hip and knee arthroplasty: a prospective, non-randomised controlled study. E-poster presentation at EWMA, 2015. 13. Bredow, J. et al. Randomized clinical trial to evaluate performance of flexible self-adherent absorbent dressing coated with silicone layer after hip, knee or spinal surgery in comparison to standard wound dressing. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 14. Eastburn, S. et al. A review of blisters caused by wound dressing components: can they impede post-operative rehabilitation and discharge? International Journal of Orthopaedic and Trauma Nursing, 2016. 15. Gupta, S.K. et al. Postoperative wound blistering: is there a link with dressing usage? Journal of Wound Care, 2002. 16. Van Overschelde, P. et al. A randomised controlled trial comparing two wound dressings used after elective hip and knee arthroplasty. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 17. Bredow, J. et al. Randomized clinical trial to evaluate performance of flexible self-adherent absorbent dressing coated with silicone layer after hip, knee or spinal surgery in comparison to standard wound dressing. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 18. Zarghooni, K. et al. Effect of a modern dressing compared to standard dressings on outcome after primary hip and knee arthroplasty: a prospective, non-randomised controlled study. E-poster presentation at EWMA conference, London, United Kingdom, 2015. 19. Feili, F. et al. A laboratory valuation of the fluid retention properties of post-operative absorbent dressings. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 20. Feili, F. et al. Fluid handling properties of post-operative wound dressings. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016. 21. Feili, F. et al. Fluid handling properties of antimicrobial post-operative wound dressings. Poster presentation at 5th Congress of the WUWHs, Florence, Italy, 2016.

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